



Equine Dental Speculum Light

Installation instructions :

1. Look for a suitable place on the **left** side of the speculum (horse) to mount the Magnet Holder.
2. Degrease the metal using the alcohol wipe.
3. Dry the metal using paper towel.
4. Make sure the metal is warmer than 15° C (60°F). If this is not the case, heat the metal and sticker using a hair blower.
5. Remove the sticker on the back of the Magnet Holder.
6. Stick the Magnet Holder into place and push firmly.
7. If you have a speculum with only round edges and no flat parts, it is advised to use extra tie wrap and/or tape.
8. "Click" the Battery Pack on the Holder by turning it into place. **Attention for the fingers, these are strong magnets, always approach from the side.**
9. Open the speculum and put the Lamp into place.
10. Route the cable so that it cannot be crushed and plug in the connector
10. The Light is now ready for use. See user Manual

Note: this device is for professional veterinarian use only. Irresponsible use may result in damage to the device or in worse case injury to the user. In any case of doubt contact the manufacturer. The manufacturer cannot be held liable in case of misuse. Info@MiCHbvba.be